
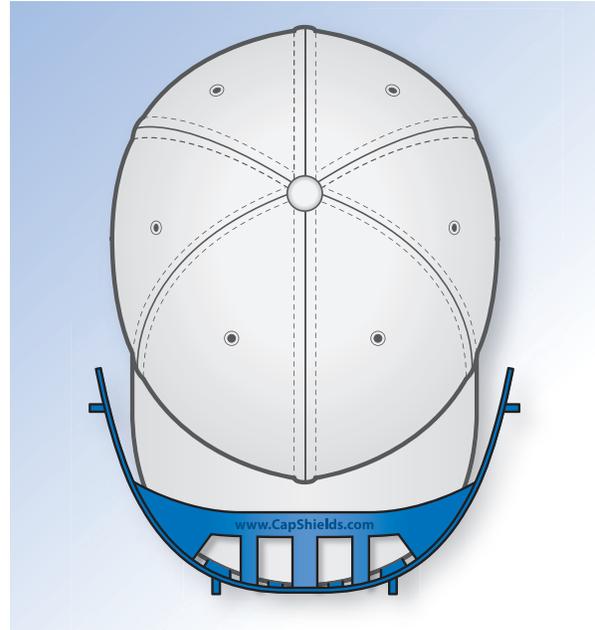


CapShields™ is a game changer for anyone who needs instant protection from easily spread germs.

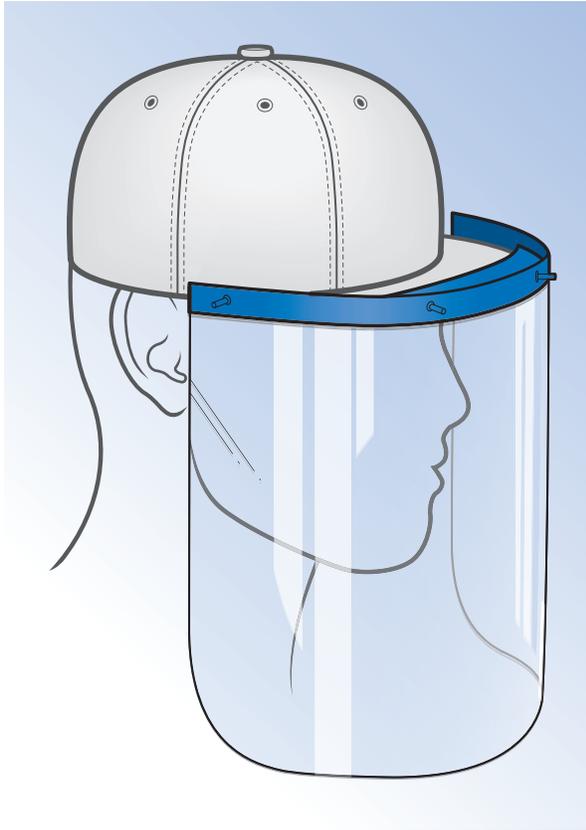

PROTECTION IN SECONDS!

With this approach anyone can instantly provide the necessary “distancing mechanism” at an affordable price.

With a simple clip, the face shield easily attaches to the bill of most baseball caps and prevents droplets and aerosol from depositing on the user's face. *CapShields do not include the baseball cap.*

In addition to medical personnel, many people in “front line” industries such as delivery personnel, take out restaurant workers, and anyone who needs to work within the proximity of others require the ability to “distance themselves” in relatively close proximity.

This is best accomplished by wearing CapShields, which provide an optically clear barrier and prevents aerosol and droplet contact with the users' face.



Order at www.capshields.com

CapShields do not include the baseball cap.
Orders prepaid via valid credit card and PayPal.



THE CURRENT PROBLEM:

Because typical face shields require an adjustable headband with a foam barrier to prevent inadvertent droplet contamination and a method of head size adjustment, they are complicated and more costly than need be for these applications.

THE CAPSHIELDS SOLUTION

Given that most people in the aforementioned professions own and wear baseball caps, it became apparent that the cap itself can serve as an ideal base for a simple and low-cost face shield.

GENERAL PRODUCT DESCRIPTION

CapShields consists of a plastic molded visor clip that attaches directly to the front of the cap bill. Attached to the clip is a clear plastic face shield that provides full-face coverage.

Molded into the clip is the company information, website and phone number for reordering.

CapShields is sold unassembled and in a sealed plastic bag along with an instruction card. This provides for a flat package that can be easily shipped singularly or in multiples.

The face shield snaps easily into the clip.

ORDERING INFORMATION

P/N CV0010-1	CV-19 Face Shield with Clip	\$12.95 each
P/N CV0011-1	CV-19 Replacement Face Shield	\$ 5.95 each
	Available in Packs of 1, 5 or 10	

(CapShields do not include the baseball cap)

Orders prepaid via valid credit card and PayPal.

Orders fulfilled by: Acera, LLC.
100 Cummings Center, Suite 439C
Beverly, MA 01915



Questions/Customer Service: 978-998-4281



Order at www.capshields.com

CapShields do not include the baseball cap.
Orders prepaid via valid credit card and PayPal.






CapShields Instructions


Step 1:

Remove the visor clip and the shield from the bag.

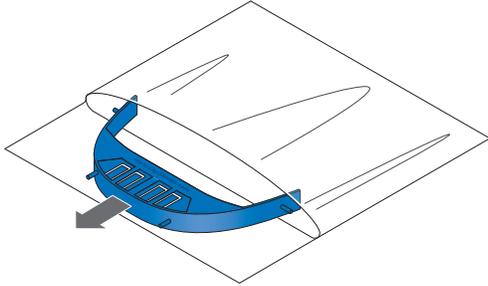

Step 2:

Remove the protective plastic from one side of the shield. **This will be the inside of the shield when in place.**

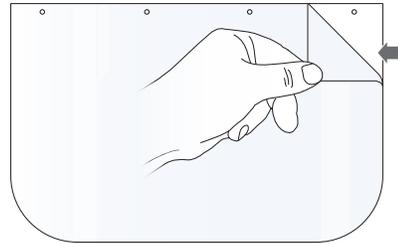

Step 3:


Making sure the clip is held so the printing is at the top and starting at one end of the clip, insert the post into the end hole of the shield.

3.1: Make sure the inside of the shield has the protective sheet **removed** and the protective sheet remains on the outside. **Do not remove the outside protective sheet yet.**

3.2: Work your way around the front of the clip, progressively inset the posts until they are all inserted into the holes. The holes are intentionally elongated so be sure to pull the shield as you work your way around so it does not slip off the post.

3.3: Smooth out the face shield so it sits firmly on the front edge of the visor clip.

3.4: You should now have the shield attached to the clip with the printing on the top.

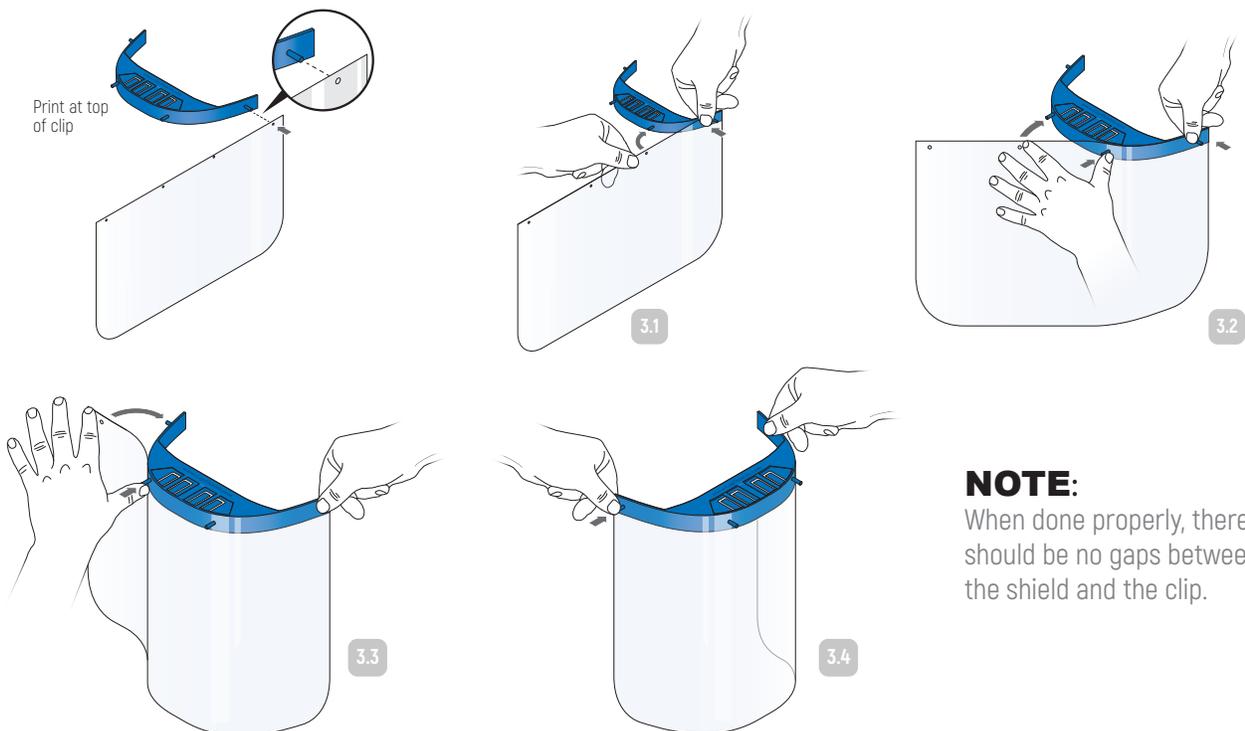


NOTE:

When done properly, there should be no gaps between the shield and the clip.


Step 4:

Select a cap. If the visor is not flat and is significantly curved, it must be straightened out in order to put the visor clip in place. Try and select a cap with the flattest bill possible to ease installation.

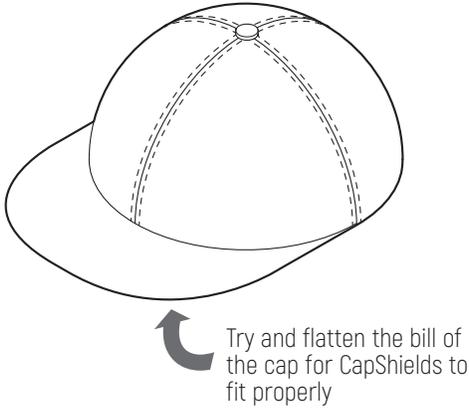


Step 5:

Slide the visor clip over the front of the bill of the cap until it is fully in position. Take care to minimize the gap between the visor clip and front edge of the bill as an open gap may allow aerosol or droplets to fall through.

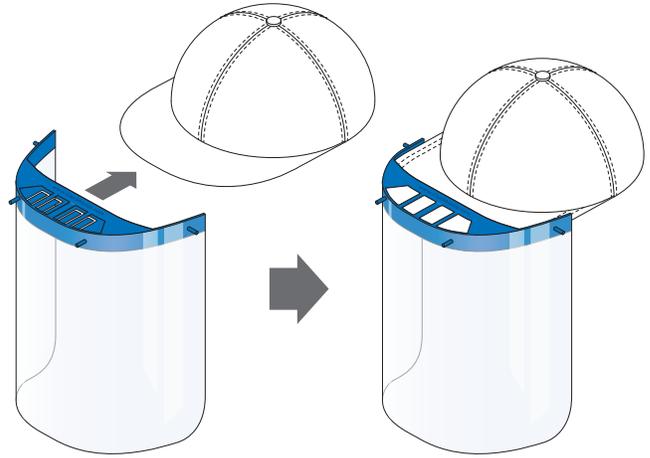

Step 6:

Carefully remove the outside protective plastic.

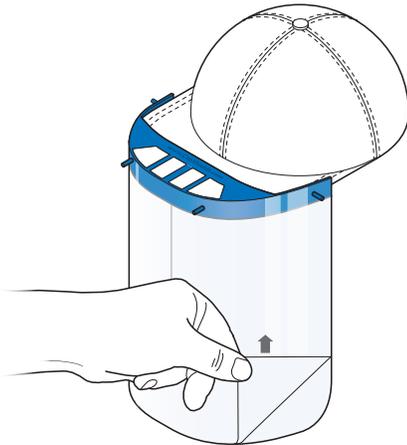


NOTE:

Fogging can occur while wearing the CapShields in cold weather. Use an anti fogging spray if this happens. Do not use if vision is compromised.

Step 7:

Adjust the headband of the baseball cap for a comfortable fit.

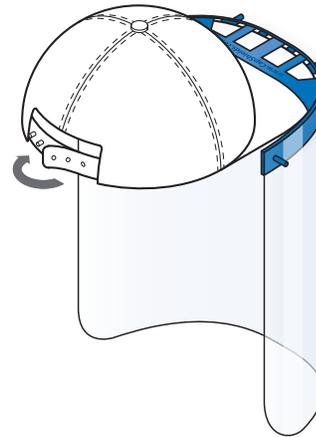

ORDERING INFORMATION

- | | | |
|--------------|----------------------------------|---------------------|
| P/N CV0010-1 | CV-19 Face Shield with Clip | \$12.95 each |
| P/N CV0011-1 | CV-19 Replacement Face Shield | \$ 5.95 each |
| | Available in Packs of 1, 5 or 10 | |

(CapShields do not include the baseball cap)

For additional ordering information visit www.capshields.com

